

Universal Buccal Swab Collection Kit

Buccal (cheek) swabs have been added to our testing services to ensure easy and convenient testing experiences for you and your patients. Buccal swabs can only be used for genetic testing.

This swab collection kit can be used for the following genetic tests:

CICA: Celiac, IBS, and Crohn's Array (Genetic Only)

The CICA (Genetic Only) evaluates an individual's risk factors for celiac disease, IBS, and Crohn's disease.

- HLA - DQ 2.5
- HLA - DQ 8
- ATG16L1
- NOD2

Methylation Detoxification Profile (Genetic Only)

The MethylDetox profile makes it easy to understand the complicated methylation process by giving comprehensive insights into the functional status of the methylation pathway.

Advanced MethylDetox (Genetic Only)

- MTHFR
- MTR
- COMT
- MTRR
- AHCY

Basic MethylDetox (Genetic Only)

- MTHFR
- MTR
- COMT



852 S. Military Trail, Deerfield Beach, FL 33442 | 800.872.5228 | info@cellsciencesystems.com

610-510-REV A

Swab Collection Instructions

1. Do not eat or drink at least 30 minutes prior to collection.
2. Wash hands thoroughly prior to collection.
3. Make sure to collect both (2) swabs separately.

1. Complete CSS Req Form


Complete the requisition form. Make sure all information requested is provided. (Don't forget to include date and time of collection)

2. Prep swab



Remove swabs from packaging. Open the plastic cap and slide the tip protector down the swab stick. **DO NOT** touch the cotton tip with your fingers

3. Obtain Sample

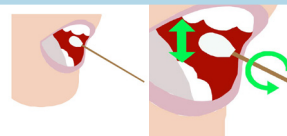

Using a downward motion as you rotate the swab stick between your fingers, rub the swab **FIRMLY** inside the cheek about six (6) times. Focus on collecting cheek cells, **NOT saliva**

4. Seal & Repeat



Slide the tip protector back up the swab stick and close the cap firmly.

Repeat steps 2-4 for the second swab

5. Swab Return Envelope

Write patient NAME, DATE and TIME of collection on the outside of the swab return envelope. Place both swabs inside and seal.

6. Shipping Envelope



Insert sealed swab return envelope & requisition form into this padded envelope then place the padded envelope into the shipping mailer.