

# SHARE YOUR CGM DATA WITH YOUR CLINIC

## USE DEXCOM CLARITY® TO UNDERSTAND GLUCOSE PATTERNS AND TO ANALYZE OR SHARE DATA WITH YOUR CLINIC

### Receive an invitation to share your data.

Your clinic will print or email an invitation to you.

### Share data with your clinic

Smith Diabetes Care LLC invites you to share your data using Dexcom CLARITY®.

Dexcom CLARITY software captures your continuous glucose monitoring (CGM) data so you and your clinic can view patterns, trends and statistics anytime, anywhere.

Your sharing code\*

**TJZQ-SVQB-CPYR**

1 Enter this code at <https://clarity.dexcom.com/share>

\*Expires: 11/30/2017

### Option 1: Share using the CLARITY website

- 1 Click <https://clarity.dexcom.com/share> or enter it into your internet browser. Confirm to share before the expiration date.
- 2 Click [Log in](#).
- 3 Enter your username and password, then click [Sign in](#).  
Log in with your Dexcom or G5 Mobile account.
- 4 Enter the sharing code from your invitation.
- 5 Enter your date of birth.
- 6 Click [Continue](#).
- 7 Click [I consent to share my data with my clinic](#).
- 8 Click [Yes, Share My Data](#).

### Share Data with a New Clinic

Your clinic will provide a sharing code for your data so that they have continuous access and can make your appointments more efficient. If your clinic hasn't provided you an invitation to share, you can ask your clinic to create one for you. You can stop sharing with your clinic on at any time.

Enter the sharing code provided by your clinic

4

Verify your date of birth

5 Month  DO  YYYY

6 [Continue](#) [Cancel](#)

### Share Data with a New Clinic

Please confirm your healthcare clinic's name below.

 **Smith Diabetes Care LLC**

7 ☒ I consent to share my data with my clinic.

8 [Yes, Share My Data](#) [Cancel](#)

#### SAFETY STATEMENT

The web-based Dexcom CLARITY® software is intended for use by both home users and healthcare professionals to assist people with diabetes in the review, analysis, and evaluation of historical CGM data to support effective diabetes management. It is intended for use as an accessory to Dexcom CGM devices with data interface capabilities. Caution: The software does not provide any medical advice and should not be used for that purpose. Home users must consult a healthcare professional before making any medical interpretation and therapy adjustments from the information in the software. Caution: Healthcare professionals should use information in the software in conjunction with other clinical information available to them. Caution: Federal (US) law restricts this device to sale by or on the order of a licensed healthcare professional.

For help with CLARITY, contact Dexcom Support at:  
**1-844-607-8398** or [techsupport@dexcom.com](mailto:techsupport@dexcom.com)


©2016-2017 Dexcom Inc. All rights reserved.

**dexcom**  
**CLARITY®**  
LBL013937 Rev 007 MT23871



## Option 2: Share using Dexcom CLARITY Reports app

- 1 Install **Dexcom CLARITY Reports** app on your smart device.
- 2 Log in with your Dexcom account.  
If you use Dexcom G5 Mobile, you'll be logged in automatically.
- 3 Tap **Share**.
- 4 Tap **Accept Invitation**.
- 5 Enter the sharing code provided by your clinic.
- 6 Tap **Enter Birthday** to select your date of birth.
- 7 Tap **Continue**.
- 8 Tap the **I consent to share my data with my clinic** box.
- 9 Tap **Yes, Share My Data**.

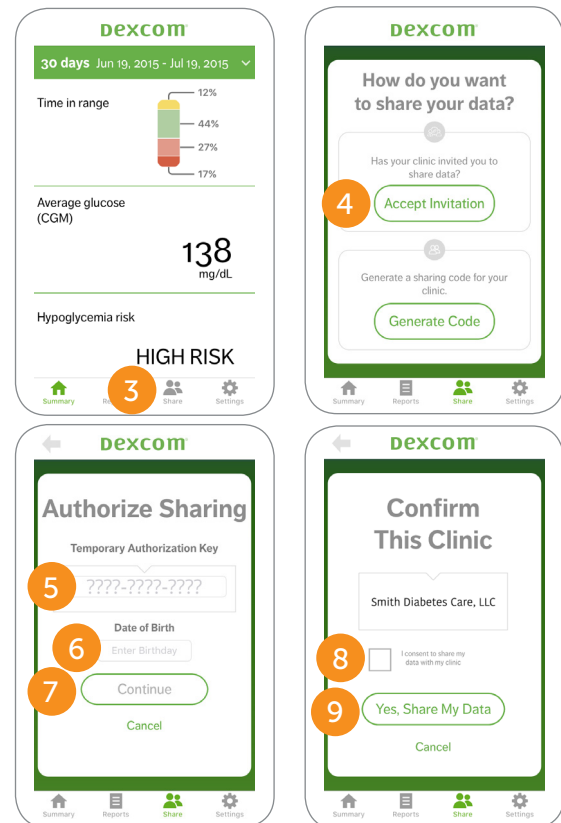

## To stop sharing with a clinic:

- 1 Enter <https://clarity.dexcom.com> into your internet browser.  
You must log in to the full CLARITY software to stop sharing.
- 2 Click **Settings** at the top of the page.
- 3 Click **Stop Sharing with this Clinic**.

