

# Virtual Visit Set Up with Dexcom CLARITY App




## Authorize sharing with your clinic so they have access to your data during face to face and remote visits

Managing diabetes can be complex, but Dexcom CLARITY can make virtual diabetes reporting simple. Interactive reports can be viewed, saved and printed for efficient clinic appointments!

By downloading and using the cloud based Dexcom CLARITY app, your data is automatically and continuously being uploaded. Your health care team then accesses Dexcom CLARITY via a web-based platform that allows them to securely view your data from anywhere.

## Eliminate the upload hassle!

View, save, print & share interactive reports for secure, efficient remote monitoring appointments with your healthcare professional. No need to upload your data.

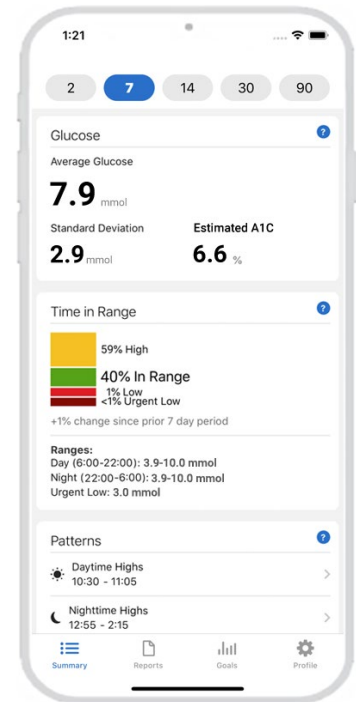

**STEP 1** Download the Dexcom CLARITY app on your mobile device.\*




**STEP 2** Log in using the same credentials as your Dexcom account.

**STEP 3** Review your key glucose metrics before your clinic appointment.


## Share Data with your Clinic

- Tap Profile
- Tap Authorize Sharing
- Tap accept invitation
- Enter the Sharing Code provided by your clinic
- Select your Date of Birth
- Tap Continue
- Tap the 'I consent to share my data with my clinic'
- Tap Yes, Share My Data

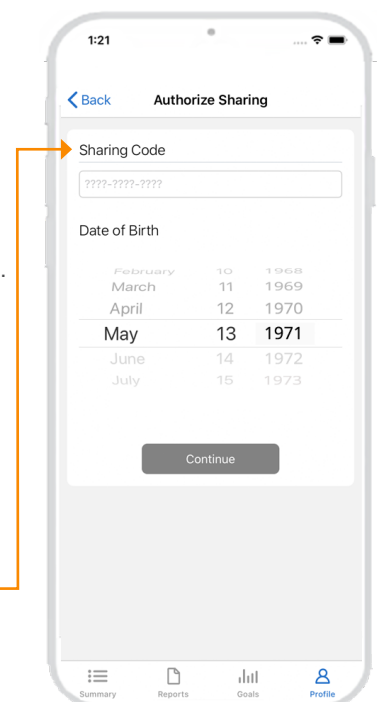


\* For a list of compatible devices, visit [dexcom.com/compatibility](https://dexcom.com/compatibility).

## Share, print and save your reports

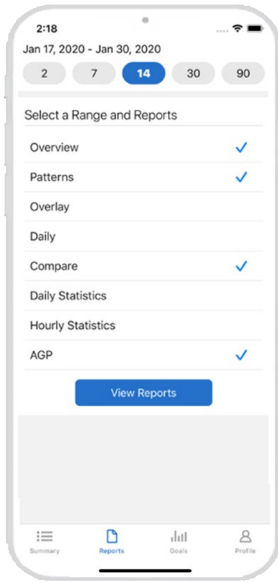


1. Tap on Reports
2. Select the Date Range and Reports you'd like to see
3. Tap View Reports

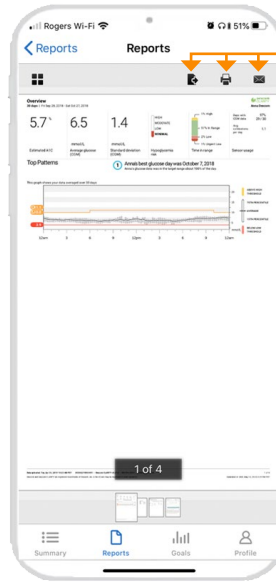


Here you can save, print or email the report to your health care professional.

## Opt-in for daily and/or weekly notifications and summary emails of your progress

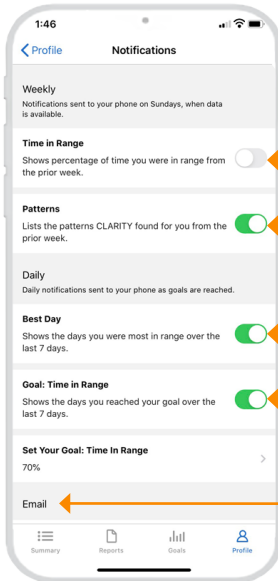

1. Tap Profile
2. Tap notifications
3. Toggle on/off:
  - Time in Range
  - Patterns
  - Best Day
  - Goal: Time in Range
  - Email summary

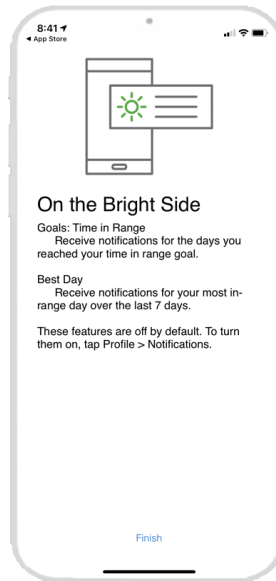

Select a daily percent time in range (%TIR) goal. Receive positive alerts from CLARITY each day that goal is reached. Opt-in to receive "Best Day" notifications, based on the CLARITY's assessment of %TIR over the last 7 days.

## Need assistance? We're here to help.

### Technical Support

Available 24/7 for product troubleshooting and CLARITY software support.

**1-844-832-1809**

### Dexcom CARE

Training & support for new Dexcom users from a team of Certified Diabetes Educators and Product Specialists.

**1-844-832-1811**

Mon-Fri 9am - 5:30pm EST†

### Customer Service

Assistance with Dexcom orders or general inquiries.

**1-844-832-1810**

Mon-Fri 9am - 5:30pm EST†

† Hours are subject to change and exclude holidays.