



Dexcom CLARITY software presents relevant patterns and trends to help make diabetes management decisions. Create patient accounts to automatically receive continuous glucose monitoring (CGM) data that you can view interactively or print during appointments.

CLINIC SETUP

Access CLARITY on the Internet

1. Enter **<https://clarity.dexcom.eu/professional>** in your internet browser.
2. Click **Register Now**.

Register your Clinic

Every clinic must be registered by an authorized representative to be accessed by healthcare professionals. By registering the clinic, you become an Administrator for the CLARITY account.

1. Select your country from the dropdown, then click **Submit**.
2. Complete the registration form, then click **Submit**.
3. Click **Complete Registration** from the activation email sent to you.
4. Enter a username and password for yourself.
5. Click **Complete my registration**, then click **OK**.

Add Staff Accounts

As an Administrator, you can invite your clinic staff to use Dexcom CLARITY. New users must accept the invitation and create a username and password to activate their accounts.

1. Click **Staff Accounts**, then click **Add Staff Account**.
2. Enter the staff member's name and email address.
3. Select **Standard User** or **Administrator** to assign user privileges.
4. Click **Save** to send an activation email to the invited user.

Add a Patient

Add patients to receive continuous uploads, manually upload, share, view, print and export CGM data. Patients log into their own CLARITY account to view and share data.

1. Click **Patient List**.
2. Click **Add new patient**.
3. Enter the patient's name, date of birth and optional patient ID.
4. Click **Save**.

DATA SHARING

Invite Patients to Share Data

Data is not shared between the clinic and patients until the CLARITY invitation to share has been accepted by patients. Once accepted, Dexcom Mobile user data is automatically and continuously uploaded into the patient CLARITY account. Invitations can be printed or viewed for CLARITY app users or emailed for home computer acceptance.

1. Click **Patient List**.
2. Click the patient's name you want to invite.
3. Click **Share data**.
4. Select **Print an Invitation** to view or print, or select **Email an Invitation**.

Help Patients Authorize Sharing with the CLARITY app

For immediate data access during an appointment, have patients:

1. Install the Dexcom CLARITY app to their smartphone.
2. Log in with their personal Dexcom account information, as needed.
3. Tap **Share** and follow the onscreen instruction.

The Sharing Code is provided on the invitation in the section above.

Minimum requirements: dexcom.com/clarity-requirements

Dexcom CLARITY is not yet available for all countries.

For questions about CLARITY, contact your local Dexcom representative.


Intended Use/Safety Statement

The web-based Dexcom CLARITY[®] software is intended for use by both home users and healthcare professionals to assist people with diabetes in the review, analysis, and evaluation of historical CGM data to support effective diabetes management. It is intended for use as an accessory to Dexcom CGM devices with data interface capabilities. Caution: The software does not provide any medical advice and should not be used for that purpose. Home users must consult a healthcare professional before making any medical interpretation and therapy adjustments from the information in the software. Caution: Healthcare professionals should use information in the software in conjunction with other clinical information available to them.

©2018 Dexcom Inc.



Dexcom, Inc.
6340 Sequence Drive, San Diego, CA 92121 USA
www.dexcom.com
Outside US: Contact your Dexcom representative

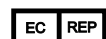

MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



LBL016216 Rev 001 MT25212