

Model No. TBSC79020

Model No. TBSC79920

Table of Contents

Important Precautions	2
Before You Begin	3
How to Set Up the Scooter	4
How to Operate the Scooter	6
Maintenance and Troubleshooting	8
Part List	11
How to Order Replacement Parts	11
Limited Warranty	Back Cover

CAUTION

Read all warnings and instructions in this manual before using the scooter. Keep this manual for future reference.

Skoot-Eze

User's Manual



Important Precautions


⚠ WARNING: To reduce the risk of serious injury, read these warnings before using the scooter.

1. Read, understand, and follow all warnings and instructions on the scooter and in this manual. Save this manual for future reference.
2. It is the owner's responsibility to ensure that all users of the scooter are adequately informed of, and follow, all warnings and instructions.
3. Inspect the scooter and tighten or replace any loose or missing parts before each use. In addition, make sure that the wheels rotate freely; the wheel bolts are tight; the horn, turn signals, mirror, and front light function well; and the brake is properly adjusted and working.
4. Keep the tires properly inflated. Do not use the scooter if the tires are underinflated; this can cause loss of control and can damage the motor or batteries.
5. Always wear protective equipment, including a helmet, knee pads, wrist guards, elbow pads, and closed-toed shoes, when using the scooter.
6. Keep children away from the scooter at all times. Always turn the key to the off position and remove it when the scooter is not in use. Always store the scooter in a protected area where unauthorized use is prevented.
7. The scooter should not be used by children.
8. Never ride the scooter on streets, roadways, or driveways that enter vehicle traffic routes. Always comply with all local laws and regulations. Use the turn signals before turning.
9. The scooter is not recommended for riders weighing more than 300 pounds. To prevent serious injury or damage to the scooter, do not exceed this maximum user weight limit.
10. Never ride the scooter on wet or uneven surfaces or when visibility is poor. Use the front light in darkened conditions.
11. Do not allow more than one person to ride the scooter at a time. Never tow another person or vehicle behind the scooter. Never allow the scooter to be towed behind a bicycle, motorcycle, or vehicle.
12. Never place hands near moving parts when the power is on. Make sure the on/off switch is in the off position before performing any of the maintenance or troubleshooting procedures described in this manual.
13. Do not ride the scooter on unpaved surfaces. Do not leave the scooter in dusty, wet, or rainy areas. Clean the scooter after each use with a damp cloth and a small amount of mild detergent. Keep the batteries and electronics dry and protected from dirt and contaminants. Charge the scooter only in a dry, well-ventilated area and follow all instructions.
14. Ride the scooter only in the restricted speed setting until you have familiarized yourself with the operation of the scooter. Always use the restricted speed setting when riding the scooter indoors or when riding the scooter in reverse.
15. The scooter is capable of high speeds, even when the motor is first started. Adjust the speed in small increments to avoid sudden jumps in speed.
16. Use only the supplied charger to charge the scooter. Do not use the charger or the scooter if any wires or electric components become damaged or frayed. Use only components or parts supplied by the manufacturer for all repairs and replacements.
17. Never remove the nuts holding the rims together without first deflating the tires.

Before You Begin

Thank you for selecting the new SCOOT-EZE™ motorized scooter. The SCOOT-EZE motorized scooter combines quiet electric power with a sturdy, compact design to provide you with comfortable motorized transportation. **For your safety and benefit, carefully read all of the warnings and instructions in this manual before using the scooter.**

Before reading further, please take a moment to familiarize yourself with the parts of the scooter that are labeled in the drawing below.

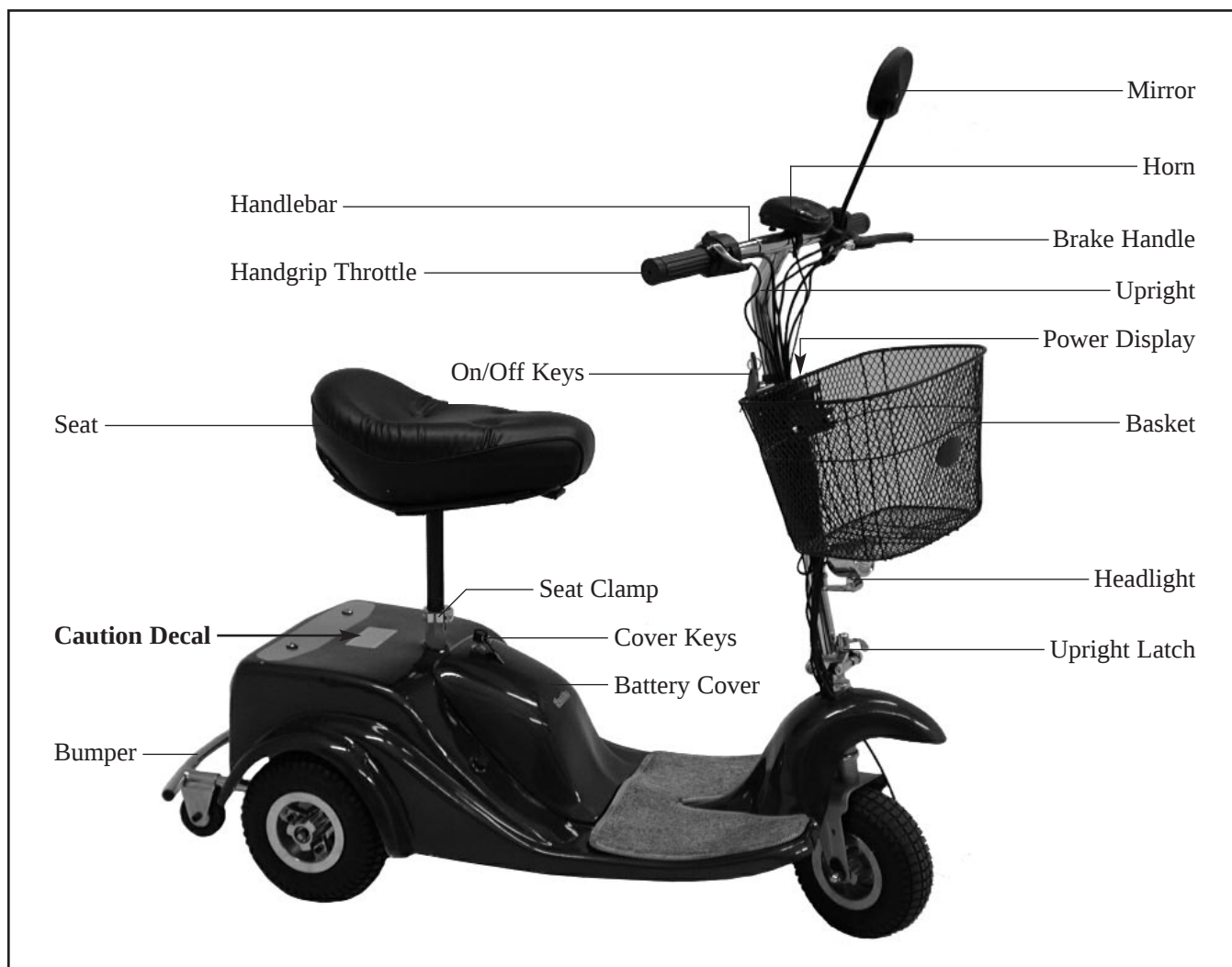

A caution decal has been placed on the scooter in the location shown above. If the decal is missing or illegible, please call our Customer Service Department toll-free at 1-800-999-3756, Monday through Friday, 6 a.m. until 6 p.m. Mountain Time, to order a free replacement decal. Apply the replacement decal in the location shown.

How to Set Up the Scooter

Follow the steps below to set up the scooter for use. **Assembly requires the included tool kit.**

1. See the inset photograph. Press and hold down the Upright Latch (19). While holding down the Upright Latch, fully raise the Upright (41).

Raise the Upright Latch (19), making sure that the latch pins snap securely into the holes in the bracket on the Upright (41). Securely fasten the Upright Latch by folding down the lever on the Upright Latch to the position shown.

2. Insert the Right Handlebar (26) into the crossbar on the Upright (41) as shown. Attach the Right Handlebar by tightening a Handlebar Screw and Handlebar Washer (22) into the outer hole in the crossbar.


Attach the Left Handlebar (27) to the crossbar in the same manner.

See the inset photograph. Remove the screw (not shown) from the indicated hole. Tighten the Mirror (24) into the hole until the Brake Handle (28) is held securely.

3. Insert the Bumper (8) into the indicated tubes on the frame. Note: It may be necessary to use a rubber mallet to fully insert the Bumper.

Tighten a Bumper Screw (not shown) into the indicated hole in each of the tubes on the frame.

4. Open the lever on the Seat Clamp (3). Insert the Seat Post (2) into the frame. Close the lever on the Seat Clamp.

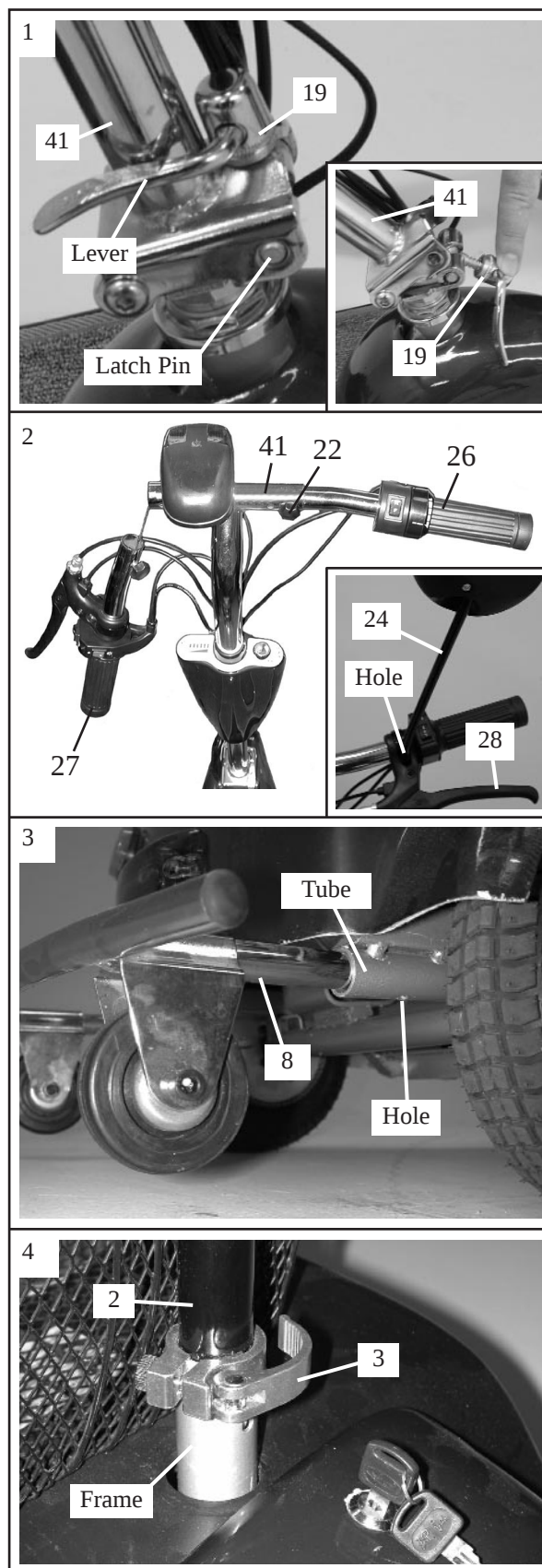

5. Make sure that the on/off key is removed from the on/off switch (see USING THE ON/OFF KEY on page 6). Unlock and open the battery cover (not shown).


Locate the plastic fuse holder. See the inset photograph. Press the long tab in the center of the fuse holder, and open the fuse holder. Remove the fuse from the small plastic clips inside of the fuse holder. Slide the ends of the red wires onto the ends of the fuse, and then press the ends of the red wires into the large plastic clips inside of the fuse holder. Close the fuse holder. Note: An extra fuse can be stored in the fuse holder.

Close and lock the battery cover.

6. The Basket (4) can be attached to either the rear of the scooter or the front of the scooter. To attach the Basket to the rear of the scooter, follow the instructions in this step. To attach the Basket to the front of the scooter, refer to step 7.

Place the Basket (4) in the position shown. Hold the Saddle Bracket (7) around the Seat Post (2) as shown in the inset photograph, and hold a Flat Bracket (43) on the opposite side of the Basket. Attach the Flat Bracket and the Basket to the Saddle Bracket with the two Saddle Bracket Bolts (5) and the two Saddle Bracket Nuts (6).

7. Locate the two black posts on the front of the Upright (41). Hold a Flat Bracket (43) against the two posts. Hold the Basket (4) in the position shown, and hold another Flat Bracket (43) on the opposite side of the Basket. Attach the Flat Brackets and the Basket to the two posts with the two Flat Bracket Screws (44).

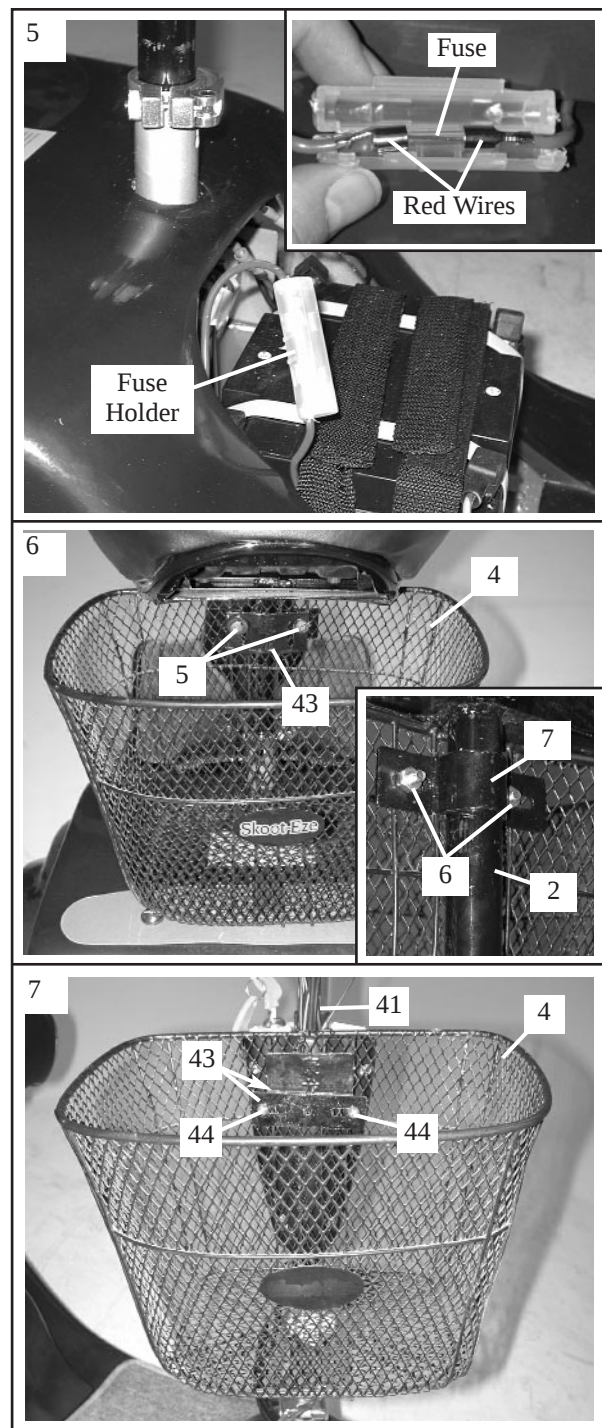

IMPORTANT: Before using the scooter, make sure that the scooter is set up as described and that all parts are securely tightened.

The batteries must be charged before the scooter can be used. Refer to CHARGING THE BATTERIES on page 6 for instructions.


How to Operate the Scooter

Important: Read all instructions in this section before using the scooter.

CHARGING THE BATTERIES

The scooter's sealed lead-acid batteries should be charged and stored at temperatures between 50°F and 100°F (10°C and 38°C). If the temperature is warmer, the batteries will discharge more rapidly. If the temperature is cooler, the scooter should be moved to a warmer area while the batteries are being charged. Follow the steps below to charge the batteries.

1. Make sure the on/off key has been removed before recharging the battery. **Important: If the motor is operated while the batteries are being charged, the battery charger will be damaged and the scooter may also be damaged.**
2. Locate the charging socket. Identify the cord on the battery charger that fits into the charging socket and plug the cord into the charging socket. Plug the other cord on the battery charger into a 110-volt outlet. A red light on the battery charger will illuminate.

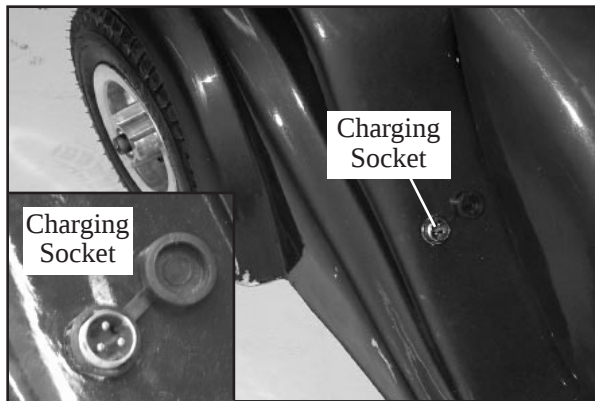


3. Allow the batteries to charge until the red light on the battery charger changes to green; the green light indicates that the battery is full. After the batteries are charged, unplug the battery charger from the 110-volt outlet **first** and then unplug the battery charger from the charging socket on the scooter.

For maximum battery life, avoid leaving the on/off key in the ON position when the scooter is not in use. Avoid draining the batteries. It is recommended that the batteries be charged after each use. The batteries should be charged at least once each month, even if the scooter is not used.

If the batteries need to be replaced, refer to HOW TO ORDER REPLACEMENT PARTS on page 11. Recycle batteries according to federal and state laws.

STORING THE BATTERY CHARGER


The battery charger can be stored on top of the batteries inside the battery compartment. To store the battery charger, unlock and open the battery cover, and place the charger on top of the batteries. Close and lock the battery cover.

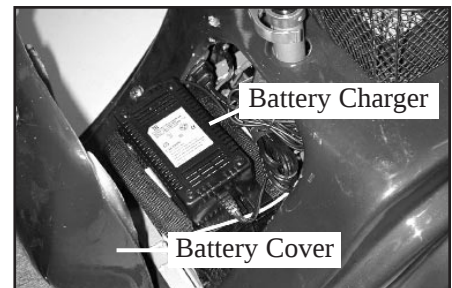

USING THE ON/OFF KEY

The on/off switch on the upright is the main power switch. Inserting the on/off key and turning it to either the "ON" or the


"R" position will turn on the power and light the power display. Each light in the power display represents 20% of the batteries' full charge. If all five lights are lit, the batteries are fully charged; if 4 lights are lit, the batteries currently hold 80% of their full charge, and so forth. Turning the key to the "OFF" position will turn off the power. **Always turn the key to the "OFF" position and remove it when the scooter is not in use.**

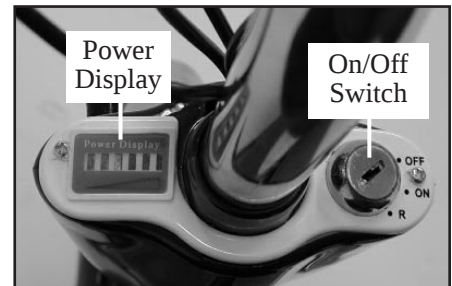

OPERATING THE SCOOTER


To use the scooter, first hold the handlebars securely, step onto the deck, and sit down onto the seat. Next, turn the on/off key to the restricted speed (“R”) position or to the “ON” position. Note: If the restricted speed setting is selected, the scooter will be restricted from reaching full speed. **Always use the restricted speed setting when riding the scooter indoors or when riding the scooter in reverse.**

The scooter can then be started with the handgrip throttle on the right handlebar. To start the scooter, securely grip the handgrip throttle and slowly roll it backward (counterclockwise). **The scooter will immediately begin to move.** Move your hand to roll the throttle **slowly** forward or backward until the scooter moves at the desired speed.

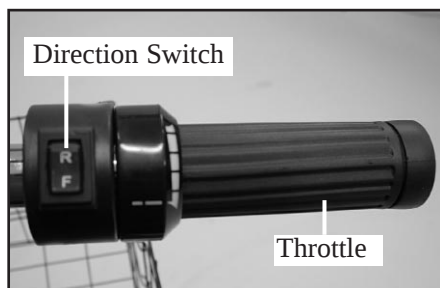

⚠ CAUTION: Use the scooter only in the restricted speed setting until you have familiarized yourself with the scooter’s operation; the scooter is capable of high speeds even when the motor is first started. Adjust the speed in small increments to avoid sudden jumps in speed.

Pull the brake handle on the left handlebar to slow the scooter quickly. When the handgrip throttle is released or the brakes are used, the motor will stop. **Important: Avoid frequently starting and stopping the motor; frequent starting reduces the batteries’ charge and may diminish battery life. Do not repeatedly roll and release the handgrip throttle in an attempt to move the scooter at a slower speed; doing this may damage the motor.**

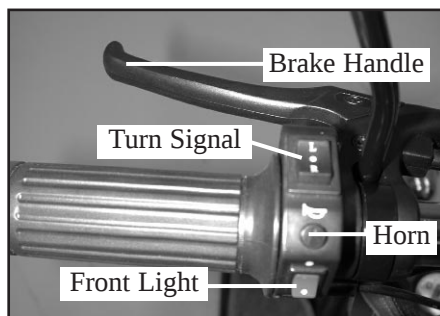

Using the direction switch on the right handlebar (see the photograph at the left), the scooter can be made to drive backward as well as forward. Press the top of the switch, marked “R,” to drive the scooter in reverse. When the switch is set to reverse, a red light in the switch will be lit. Adjust the speed with the throttle and brake as described above. To drive the scooter forward, press the bottom of the switch, marked “F.”


The left handlebar has several controls and a rear-view mirror for your convenience and safety while riding the scooter. (See the photograph at the lower left.) Use the turn signal switch and horn button to signal your intentions to others. The front light switch allows you to turn on the front light in darkened conditions or when visibility is poor. Adjust the mirror to allow for a clear view of what lies behind you.

Use the scooter only on paved, smooth, level surfaces. Avoid using the scooter on grades steeper than 5% for extended periods. **Important: Avoid starting the motor on an upward slope; starting the motor uphill strains the motor and may diminish battery life.**

The scooter is rated for use on a hard, level surface with a surrounding temperature of 50°F and a maximum user weight of 300 pounds; the batteries’ charge will be used faster by heavier riders. With the batteries fully charged, the scooter can travel up to 12 miles under optimal conditions. **Important: Make sure the tires are inflated to the pressure listed on the sides of the tires; underinflated tires will strain the motor and batteries.**

SELECTING A TONE FOR THE HORN

The horn on the handlebars can produce eight different tones. Turn the indicated selector knob to select the desired tone.

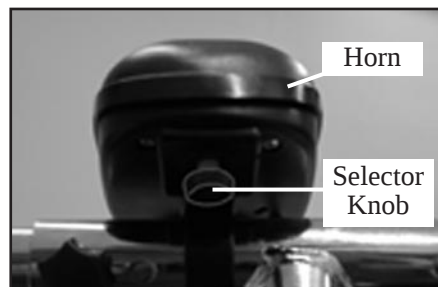

ADJUSTING THE HANDLEBARS

The handlebars can be adjusted for a closer grip. Refer to assembly step 2 on page 4 and attach the handlebars at the inner holes, rather than at the outer holes.

Maintenance and Troubleshooting

Regularly check the scooter and make sure that all parts are properly tightened. If the scooter does not function properly, follow the steps below. **IMPORTANT: Always turn the on/off key to the “OFF” position and remove it before performing any of the steps below.**

Symptom 1—The motor does not start when the handgrip throttle is turned.


1. Make sure that the on/off key is in the on/off switch and that the key is turned to either the “ON” or the “R” position.
2. Make sure that the batteries are charged.
3. Make sure that the brake handle is not being pulled.

Symptom 2—The batteries discharge quickly.

1. Make sure that the tires are inflated to the pressure listed on the sides of the tires.
2. Use the scooter on level, smooth surfaces. Avoid using the scooter on grades steeper than 5%.
3. Charge and store the batteries at temperatures between 50°F and 100°F (10°C and 38°C).
4. If the batteries still discharge quickly, new batteries may be needed. Refer to HOW TO ORDER REPLACEMENT PARTS on page 11.


Symptom 3—The brake is too loose or too tight.

1. Locate the locknut and adjustment barrel on the brake handle. Turn the locknut counterclockwise to loosen it. Next, turn the adjustment barrel counterclockwise to tighten the brake or clockwise to loosen it. Then, turn the locknut clockwise and retighten it. Repeat until the brake is properly adjusted.

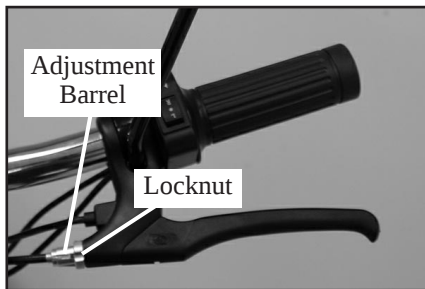


If the brake requires further adjustment, locate the locknut and adjustment barrel near the lower end of

the brake cable. Using a small open-end wrench, loosen the locknut. Next, turn the adjustment barrel counterclockwise to tighten the brake or clockwise to loosen it. Then, retighten the locknut. Repeat until the brake is properly adjusted.

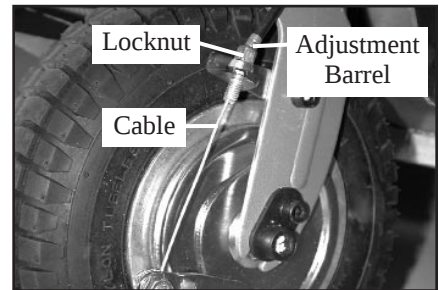

Symptom 4—The seat clamp is too loose or too tight.


1. If the seat clamp does not hold the seat post tightly, open the lever. Turn the adjustment nut clockwise one half turn while keeping the lever from turning. Close the lever. Repeat until the seat post is held tightly.

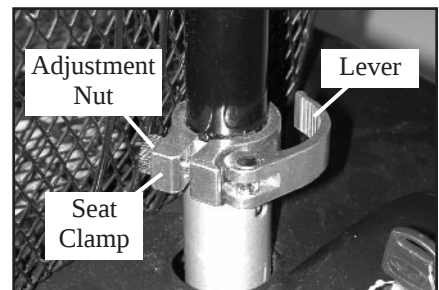

If the seat clamp cannot be closed, open the lever and turn the adjustment nut counterclockwise one quarter turn while keeping the lever from turning. Close the lever. Repeat until the seat clamp closes and holds the seat post tightly.

Symptom 5—The volume of the horn becomes weak.

1. The batteries should be replaced, using two “AA” batteries (alkaline batteries are recommended). Remove the two screws and remove the cover of the horn. Replace the batteries. Make sure to match the + and – symbols on the batteries with the symbols in the battery compartment of the horn. Reattach the cover.

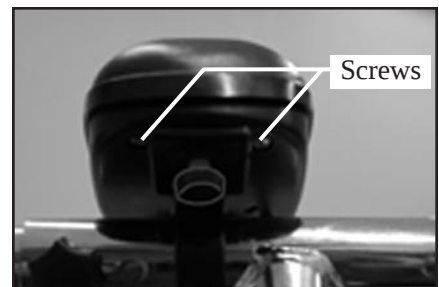

Part List—Model No. TBSC79020/TBSC79920

R0103A

Key No.	Qty.	Description	Key No.	Qty.	Description
1	1	Seat	25	2	Bumper Endcap
2	1	Seat Post	26	1	Right Handlebar w/Throttle
3	1	Seat Clamp	27	1	Left Handlebar w/Light Controls
4	1	Basket	28	1	Brake Handle
5	2	Saddle Bracket Bolt	29	1	Motor Assembly
6	2	Saddle Bracket Nut	30	3	Tire
7	1	Saddle Bracket	31	3	Tube
8	1	Bumper	32	1	Chain
9	2	Bumper Screw	33	1	Brake Cable
10	2	Safety Wheel	34	3	Front Wheel Bolt
11	2	Safety Wheel Bolt	35	3	Front Wheel Nut
12	2	Safety Wheel Nut	36	6	Rear Wheel Screw
13	3	Signal/Stop Light Bulb	37	1	Front Wheel
14	1	Signal Light Lens	38	2	Rear Wheel
15	1	Stop Light Lens	39	1	Headlight Bulb
16	2	Battery	40	1	Brake Shoe Set
17	1	Battery Charger	41	1	Upright
18	1	Carpet Mat	42	1	Tool Kit
19	1	Upright Latch	43	3	Flat Bracket
20	1	Headlight w/Bracket	44	2	Flat Bracket Screw
21	1	On/Off Key	45	1	Fuse
22	2	Handlebar Screw w/Washer			
23	1	Horn			
24	1	Mirror			

Note: Specifications are subject to change without notice.

How to Order Replacement Parts

To order replacement parts, call toll-free 1-800-999-3756, Monday through Friday, 6 a.m. until 6 p.m. Mountain Time (excluding holidays). When ordering parts, please mention the model number of the scooter (TBSC79020 or TBSC79920) and the key number and description of the needed parts.

Limited Warranty

ICON Health & Fitness, Inc. (ICON) warrants this product to be free from defects in workmanship and material, under normal use and service conditions, for a period of ninety (90) days from the date of purchase. This warranty extends only to the original purchaser. ICON's obligation under this warranty is limited to the replacement of parts only. No returns will be accepted by ICON. This warranty does not extend to any product or damage to a product caused by or attributable to freight damage, abuse, misuse, improper or abnormal usage or repairs not provided by an ICON authorized service center; to products used for commercial or rental purposes; or to products used as store display models. No other warranty beyond that specifically set forth above is authorized by ICON.

ICON is not responsible or liable for indirect, special or consequential damages arising out of or in connection with the use or performance of the product or damages with respect to any economic loss, loss of property, loss of revenues or profits, loss of enjoyment or use, costs of removal, installation or other consequential damages of whatsoever nature. Some states do not allow the exclusion or limitation of incidental or consequential damages. Accordingly, the above limitation may not apply to you.

The warranty extended hereunder is in lieu of any and all other warranties and any implied warranties of merchantability or fitness for a particular purpose is limited in its scope and duration to the terms set forth herein. Some states do not allow limitations on how long an implied warranty lasts. Accordingly, the above limitation may not apply to you. This warranty gives you specific legal rights. You may also have other rights which vary from state to state.

ICON HEALTH & FITNESS, INC., 1500 S. 1000 W., LOGAN, UT 84321-9813