

Limited Warranty

ICON Health & Fitness, Inc. (ICON), warrants this product to be free from defects in workmanship and material, under normal use and service conditions, for a period of ninety (90) days from the date of purchase. This warranty extends only to the original purchaser. ICON's obligation under this warranty is limited to replacing or repairing, at ICON's option, the product at one of its authorized service centers. All products for which warranty claim is made must be received by ICON at one of its authorized service centers with all freight and other transportation charges prepaid, accompanied by sufficient proof of purchase. All returns must be pre-authorized by ICON. This warranty does not extend to any product or damage to a product caused by or attributable to freight damage, abuse, misuse, improper or abnormal usage or repairs not provided by an ICON authorized service center, products used for commercial or rental purposes, or products used as store display models. No other warranty beyond that specifically set forth above is authorized by ICON.

ICON is not responsible or liable for indirect, special or consequential damages arising out of or in connection with the use or performance of the product or damages with respect to any economic loss, loss of property, loss of revenues or profits, loss of enjoyment or use, costs of removal, installation or other consequential damages of whatsoever nature. Some states do not allow the exclusion or limitation of incidental or consequential damages. Accordingly, the above limitation may not apply to you. The warranty extended hereunder is in lieu of any and all other warranties and any implied warranties of merchantability or fitness for a particular purpose is limited in its scope and duration to the terms set forth herein. Some states do not allow limitations on how long an implied warranty lasts. Accordingly, the above limitation may not apply to you.

This warranty gives you specific legal rights. You may also have other rights which vary from state to state.

ICON HEALTH & FITNESS, INC., 1500 S. 1000 W., LOGAN, UT 84321-9813


WESLO is a registered trademark of ICON Health & Fitness, Inc.

Part No.152074 H04293-C R1298A
Printed in USA © 1998 ICON Health & Fitness, Inc.

WESLO Knee Support
Wrist Support
Elbow Support

USER'S MANUAL

Model No. WLRX24080

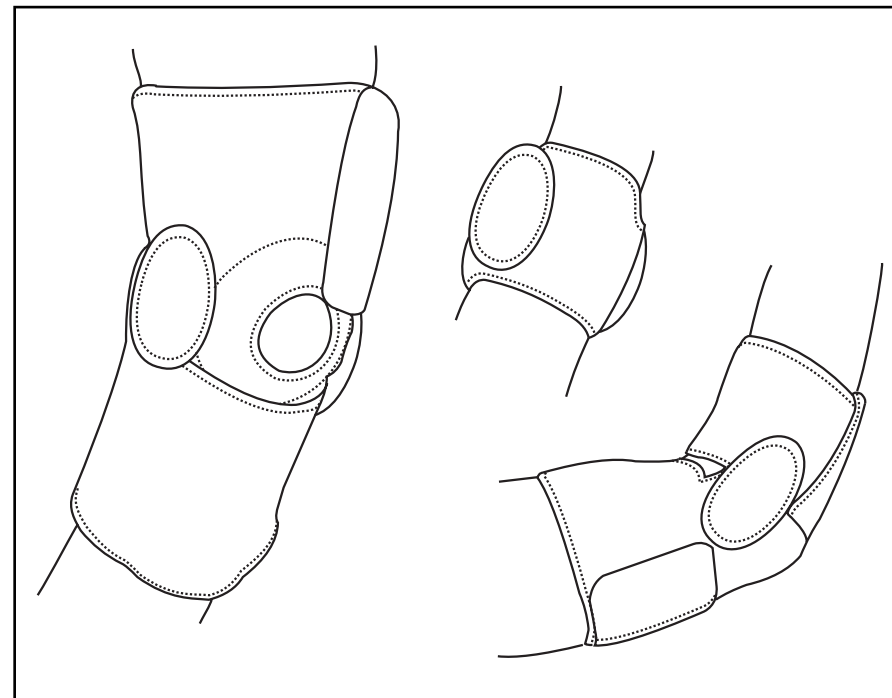

WESLO® Magnetic Therapy Supports provide therapeutic relief from chronic pain without limiting daily activities. They keep joints and muscles warm, flexible and supported with or without magnets in place.

This package contains three unifit neoprene supports: one Elbow Support, one Knee Support and one Wrist Support. Each product has two removable pouches containing four magnets each.

Important Precautions

WESLO® Magnetic Therapy Supports should never be placed directly over an open wound or transdermal drug delivery patch.

Magnetics should never be used by individuals using pacemakers, insulin pumps or other electronic implants.

Not recommended for use during the first trimester of pregnancy.

Keep all electronic devices at a distance when using magnetic therapy. They may become damaged and alter the strength of the magnetic field.

QUESTIONS?

As a manufacturer, we are committed to providing complete customer satisfaction. If you have questions, we will guarantee complete satisfaction through direct assistance from our factory.

**Please call our
CUSTOMER HOT LINE:
1-800-999-3756**

Mon.–Fri., 6 a.m.–6 p.m. MST

How to Use the WESLO® Knee Support


Wrap the top section of the support around your leg above the knee and secure it with the fastener strip.

Note: The smooth side of the support must be facing the skin.

Make sure the support is tight enough to be stable, but without being uncomfortable.

Make sure the seams of the mid-section are centered around the knee cap, then secure the bottom closure across your calf muscle. To ensure freedom of movement, the crease on the back of the support should be positioned directly over the bend on the inside of your knee.

Apply the removable pouches on opposite sides of the knee. For best results, place the pouches directly over the injured area.

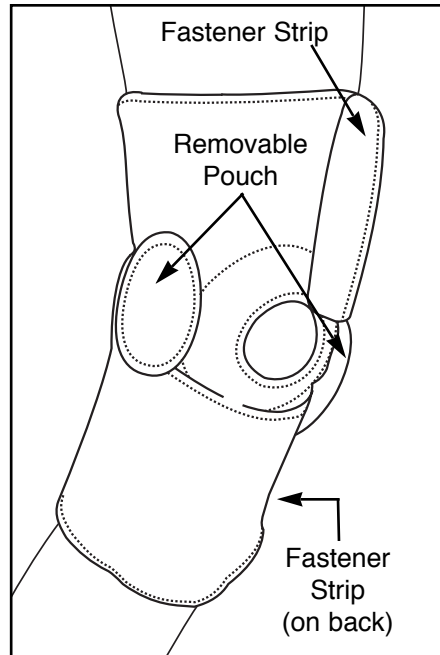

How to Use the WESLO® Elbow Support


Position your elbow on the seam in the center of the support. Wrap the bottom section of the support around your arm below the elbow and secure it with the fastener strip.

Note: The smooth side of the support must be facing the skin.

Make sure the support is tight enough to be stable, but without being uncomfortable.

To ensure freedom of movement, make sure the crease of the support is centered over the bend inside your elbow. Wrap the top closure around your upper arm.

Apply the removable pouches on opposite sides of the elbow. For best results, place the pouches directly over the injured area.

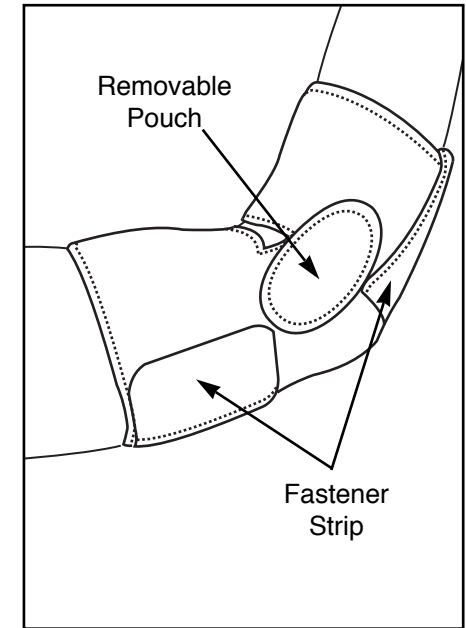


How to Use the WESLO® Wrist Support

Wrap the support around your wrist and secure it with the fastener strip.

Note: The smooth side of the support must be facing the skin.

Make sure the support is tight enough to be stable, but without being uncomfortable.

Apply the removable pouches on opposite sides of the wrist. For best results, place the pouches directly over the injured area.

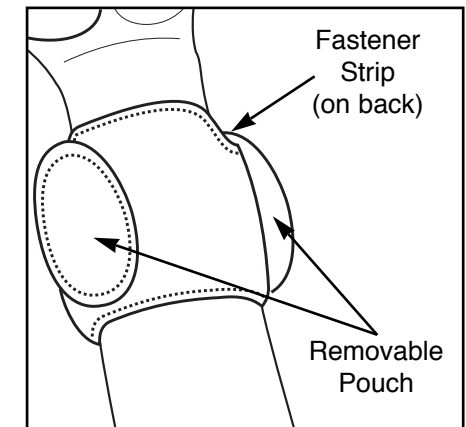

Washing Instructions

Remove the pouches containing the magnets. Machine wash the

supports in cold water. Air dry. Do not machine dry.