

iPRO™ 2 APP

STEP BY STEP GUIDE

PATIENT




Clinic ID:

1. **DOWNLOAD** the app from iTunes store or Google Play by searching **iPro 2 App**^{1,2}




2. SET UP AN ACCOUNT

Follow the steps within the app by creating a password, inputting your information, and adding the clinic ID to complete setup. You will receive the clinic ID from your healthcare professional³.

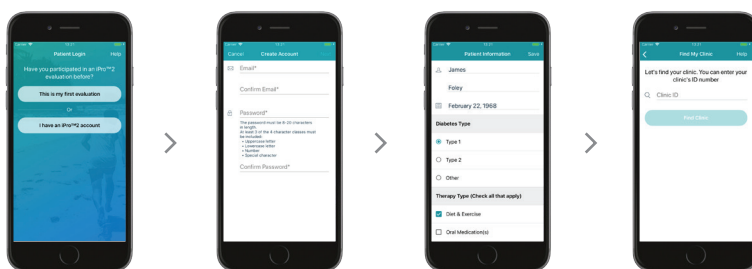


3. LOG EVENTS

- For the next 6 days, record events, such as BG, meals, medication, exercise and sleep.
 - For best accuracy, enter data immediately
 - Check and record your **BGs 3-4 times per day**, before meals and bedtime
- Events will be automatically uploaded to your CareLink™ iPro™ reports

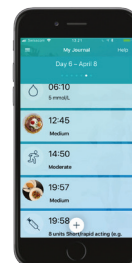

4. AT THE END OF THE EVALUATION

- Please ensure that all events are recorded in the app
- Have access to the internet to ensure all your events have been uploaded to CareLink™ iPro™ software
- Return your iPro™ 2 recorder to your healthcare professional
- Review reports with your healthcare professional
- You do not need to end the evaluation in the app. It will end automatically after 7 days.

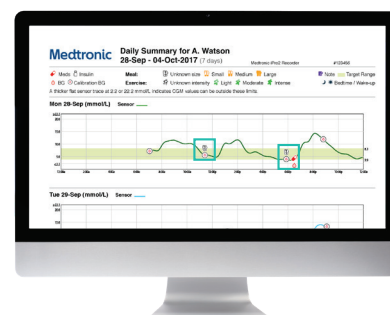


1. If you cannot download the app with your healthcare professional, the app can be setup, up to 1 day prior to evaluation start or after the evaluation start appointment. The iPro™ 2 evaluation cannot start until the app is set up. Tablets are not compatible. The iPro™ 2 evaluation cannot start until the app is set up or use a paper logbook.

2. If you forgot your password to download apps please use this information: Directions may differ based on model and operating system. App Store (iPhone): Tap Settings>tap [your name]> Password & Security > Change Password, then follow the onscreen steps to update your password. For iOS 10.2 or earlier, tap iCloud > [your name]> Password & Security > Change Password, then follow the onscreen steps; Google Play(Android): Settings>Sign in and security> Change password


3. When entering clinic ID, only include letters and numbers. Do not include the dash (-).

NAVIGATE THROUGH THE APP

Home Screen

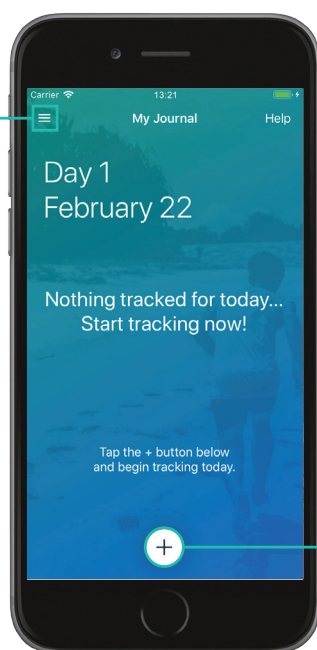

Expanded Events Menu

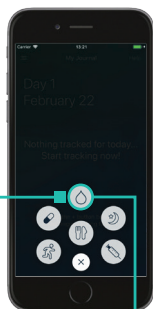

LOG IN YOUR EVENTS

BLOOD SUGAR

Record blood sugar with exact time and date

MEALS

Record meal size, time and date. Optional to record carbs and a take picture of your meal

MEDICATION

Record your diabetes medications, time and date

EXERCISE

Record physical activity intensity, time and date


SLEEP

Record sleep start time, end time and date

INSULIN

Record your insulin type, number of units, time and date


Blood Sugar

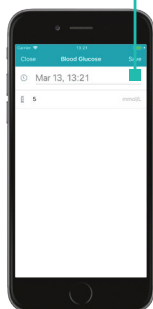

LOG BLOOD SUGAR

1. Click on the **BLOOD SUGAR** icon in the expanded events menu
2. Type in blood sugar value and confirm date and time
3. Click "Save" on the top right

Main Menu

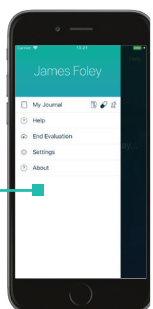

MAIN MENU OPTIONS

MY JOURNAL

See a summary of entries, add/edit events

SETTINGS

Change blood glucose units

HELP

Access answers to questions about setup, event logging, troubleshooting

END EVALUATION*

Use only if instructed by your doctor

* Allows the evaluation to be ended earlier than scheduled.

For further information, view the instructional video by searching "iPro™ 2 App" on:

<https://www.youtube.com/user/MedtronicDiabetesEU> or call our customer care team at XXXX XXXX

Medtronic

Europe

Medtronic International Trading Sàrl.
Route du Molliat 31
Case postale
CH-1131 Tolochenaz
Tel: +41 (0)21 802 70 00
Fax: +41 (0)21 802 79 00

UC201811404EE ©2018 Medtronic.
All rights reserved. Printed in Europe.

www.medtronic.eu

This information does not substitute the Instructions for Use for iPro™ 2 professional CGM.

IMPORTANT SAFETY INFORMATION: For a listing of Indications, contraindications, precautions, warnings and potential adverse events, please refer to the Instructions for Use.