



STEP 1

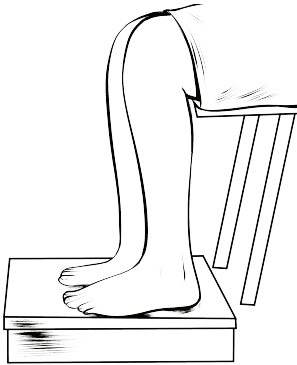


Align your patient's body properly. Establish right (90 degree) angle at hip, knee and ankle.

STEP 2

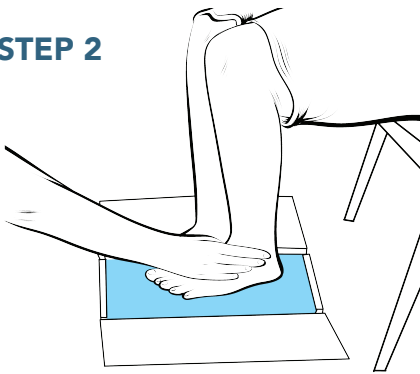


See that ankle is neither rolling out nor in.

STEP 3

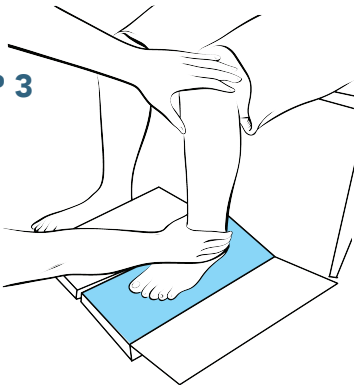


Hold ankle in neutral position, preventing ankle from rolling out or in when impression is made. Instruct the patient to apply no pressure. Place free hand on patient's knee and press down, pushing the foot 1" to 1-1/2" into foam. Simultaneously, apply pressure to opposite hand to guide the foot in straight. The ball of foot and heel should hit the bottom at the same time.

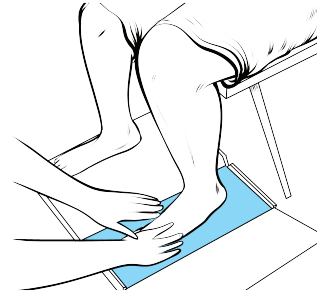

STEP 4

Finish the impression by pressing down the toes that are sticking up. It is natural for the toes to bend up while the foot is being pressed into the foam. Now have the patient stand up to ensure the foot has reached the bottom, but do not allow the ankle to roll in or out.

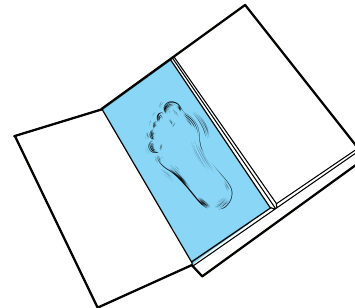

STEP 5

Inspect each impression for defects or an uneven nature of the weight bearing surface or abnormal feet contour. Repeat procedure with the other foot.

INCLUDE ORDER FORM WITH EACH CASTING KIT. MAKE SURE YOUR STORE ID# IS ON THE ORDER FORM.

Corporate Office

ARCH FITTERS

1625 SE Hogan RD Unit G

Gresham, OR 97080

LOCAL: 503-491-1723

FAX: 503-489-0706

TOLL FREE: 877-229-9652